

004426 Exhalation Port for Vivo 40, 50 & 60



Please be advised that item 004426 Exhalation Port for Vivo 40, 50 and 60, has been updated following obsolescence of the original version by the manufacturer. This also affects item 005065 Patient Circuit with Leakage (reusable) Vivo 50.

The replacement exhalation port is more robust with more stable leakage. However, unlike the previous version which offered a 15mm connection, the new version only has a 22mm - 22mm connection. **Consequently, the port will no longer connect to a 15mm HME or interface.**

Following any change to a patient breathing circuit, we recommend you always check the alarm settings, as instructed in the instructions for use. If you require any assistance, please contact us.

For further information or assistance, do not hesitate to contact us.

Tel: 01789 293460

Email: orders@nippyventilator.com