



B & D Electromedical LLP
Unit A2 The Bridge Business Centre
Timothy's Bridge Road
Stratford-upon-Avon
Warwickshire CV37 9HW
Tel: (01789) 293460
Tec Support: (01789) 721577

COUGH ASSISTORS, VENTILATORS
& BREATHING SYSTEMS

16th January 2013

IMPORTANT FIELD SAFETY NOTICE

B&D Electromedical is voluntarily advising its customers of important safety information regarding the following products.

Devices: NIPPY 3 ventilator, NIPPY JUNIOR ventilator, NIPPY ST ventilator, NIPPY S ventilator. All devices manufactured between 2003 (first production) – 2007 (last production)
This FSN does not apply to our current range of ventilators, Nippy 3+, Nippy Junior +, ST+ and S+.

Attention: All clinical staff, managers and users of the above devices.

Description of the problem: Our post market surveillance procedures have identified a potential safety risk if oxygen is entrained when using the **Nippy 3, Nippy Junior, Nippy ST or Nippy S**. The use of oxygen with these devices is not recommended as they were not designed for such use.

Potential hazard: If used with oxygen, under certain circumstances, there is a potential for a build-up of oxygen inside the machine. As oxygen supports combustion, there is a risk of fire.

Type of action: B&D Electromedical has updated the instructions for use to include a statement warning that these devices are not suitable for use with entrained oxygen. Copies may be downloaded from our website www.nippyventilator.com/downloads

Distribution of this FSN: Please transmit this notice to all those who need to be aware within your organisation or any other organisation where these devices may have been transferred.

Please return the acknowledgment form after reading this notice.

Please be aware B&D Electromedical have notified the MHRA of this action.

For more information or assistance please contact:

Mr Noel Davis - Director
B&D Electromedical LLP

A handwritten signature in black ink, appearing to read 'Noel Davis'.

e-mail at: support@nippyventilator
phone: 01789 293460



B & D Electromedical LLP
Unit A2 The Bridge Business Centre
Timothy's Bridge Road
Stratford-upon-Avon
Warwickshire CV37 9HW
Tel: (01789) 293460
Tec Support: (01789) 721577

COUGH ASSISTORS, VENTILATORS
& BREATHING SYSTEMS

Confirmation of receipt of field safety notice

Please complete the section below and return it to B&D Electromedical

Date

Name

Position.....

Department

Hospital / Organisation:

Address.....

I confirm that I have read this field safety notice and transmitted it to all persons concerned.

Signed.....